English

Plantilla RhythmLink desechable

Usos previsto

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Disposable RhythmLink Template

Intended Use

This template is intended for application as an EEG electrode accessory to assist in the placement of electrodes.

Caution

Disposable templates are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product.

Indications for Use

This template is to be used in hospitals and professional health care facilities for applications such as EEG, EP (evoked potentials), IONM (intraoperative neurophysiological monitoring), PSG (polysomnography) and ambulatory.

Instructions for Use

Verify that the patient's head size corresponds to the size range indicated on the template packaging. Position the template on the patient's head so that the "N" tab is centered between the eyes at the nasion, the circumferential strap is just above the ears, and the strap junction at the back of the template is at the inion. Fasten the chin strap and wraparound strap so that a secure fit is achieved. When finished, remove template and discard it. Single patient use only.

RhythmLink International, LLC
1140 First Street South
Columbia, SC, USA 29209-3540
+1.866.633.3754 [toll-free]
+1.803.252.1222
+1.803.252.1111 [fax]
sales@rhythmLink.com
RhythmLink.com

Australian Sponsor

Emergo Australia
Level 20 Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

FOR SINGLE USE ONLY

Rx Only

To view a list of symbol definitions found on packaging and instructions for use, please visit RhythmLink.com/symbols.

RhythmLink® is a registered trademark of RhythmLink International, LLC.

L260001 rev005